Medium Stream Hydrating Nasal Care

Isonic, sterile, 100% natural-source, desalinated, undiluted sea water without preservatives

hydraSense® Medium Stream is a simple and effective way of cleansing excessive mucus accumulation in congested nasal cavities and passages caused by allergies, colds, and upper respiratory infections.

Mucus congestion in the respiratory cavities plays a role in ear, nose and throat infections, bronchitis and other respiratory disorders in children and adults.

Composed of undiluted, sterile sea water rich in trace elements, hydraSense® relieves cold symptoms such as nasal obstruction and secretion, related sinus pressure and sore throat. Contributing as well to the effective cleansing of cold symptoms, the occurrence of complications such as sinusitis, medication use and the number of sick days, hydraSense® also reduces excessive dryness due to dust or dry air.

The specially-designed applicator, which fits the shape of the nostril, contributes to the effective cleansing of the nasal cavities and minimizes the risk of damage to the delicate mucus membrane.

Storage conditions: Store between 15°C and 30°C.

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Gently apply hydraSense® at least twice a day (morning and evening), and 4 to 6 times a day when cold or allergy symptoms are present.

hydraSense® Medium Stream does not contain propellant gases and can be used regardless of the position of the container.

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* Ivo Stapak, MD; Jana Skoupá, MD; Petr Štrnád, MD; Pavel Horák, MD. Efficacy of Isonic Nasal Wash (Seawater) in the Treatment and Prevention of Rhinitis in Children. *Archives of Otolaryngology Head Neck Surgery*, 2008; 134(1):67-74

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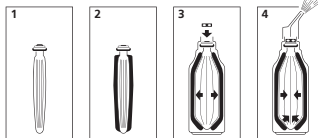
Medium Stream Hydrating Nasal Care

A sterile product without preservatives or propellant gases.

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ADVANTAGES OF THE DELIVERY SYSTEM

It's ready and easy to use:

- No mixing of solution or boiling of water required.
- The product can be used regardless of the position of the container.

It's comfortable to use:

- The isotonic sea water is at room temperature and therefore very well tolerated by the user.

It's safe:

- The nasal applicator ensures that the sterility of the product is maintained throughout its use.
- The one-way valve ensures that no mucus is transported through it to the inside of the container and therefore prevents contamination.
- Hypoallergenic.
- The removable nasal applicator allows quick cleaning.

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©Reg. TM / Marque déposée de Le Laboratoire de la Mer, 35400 Saint-Malo, France